SP-MODE® MICROSECOND LASER TECHNOLOGY





LIGHTLas TruScan Pro optimizes patient outcomes with the use of traditional Continuous Wave (CW) and our exclusive next-generation SP-Mode® Microsecond Laser Technology, included with any configuration.


Reduce Thermal Damage with Ingenious Microsecond Laser Technology: SP-Mode®


The latest innovation in LIGHTMED laser application, SP-Mode® offers a revolutionary treatment approach to achieving optimal clinical outcomes. Ongoing studies show that physicians are now able to:


- Eliminate laser-induced thermal tissue damage and treatment side effects
- Deliver a broader range of treatment modalities
- Treat disorders at a much earlier stage
- Provide safe and repeatable treatment in retinal and glaucoma applications

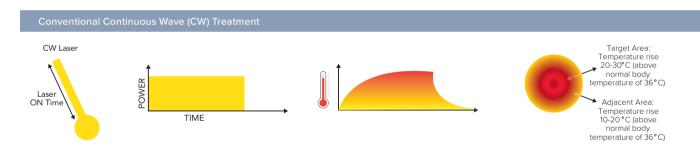


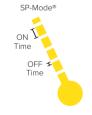

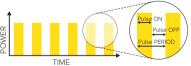


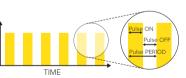

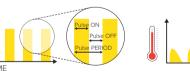

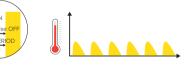



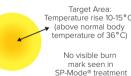

SP-Mode® Laser Trabeculoplasty (SPLT) allows significant advantages over traditional Argon Laser Trabeculoplasty (ALT) treatment with:

- Selective Photothermolysis: Targets specific cells leaving the surrounding tissue intact
- Natural Mechanisms: Stimulation of body's natural mechanisms to enhance outflow of fluid in the eye

Laser Trabeculoplasty with SP-Mode® Reduces Intraocular Pressure in Open-Angle Glaucoma

- Better Tolerance: Nearly painless treatment when compared to ALT
- Future Options: Treatment can be repeated without causing harm or furthering complications

WAVELENGTH CONFIGURATION OPTIONS

One Wavelength

Green - 532nm

Two Wavelengths Green/Yellow - 532nm/577nm Green/Yellow - 532nm/561nm Green/Red - 532nm/670nm Green/Infrared - 532nm/810nm Yellow/Red - 577nm/670nm Yellow/Infrared - 577nm/810nm

Three Wavelengths Green/Yellow/Red - 532nm/577nm/670nm Green/Yellow/Red - 532nm/561nm/670nm

Green/Yellow/Infrared - 532nm/561nm/810nm Green/Yellow/Infrared - 532nm/577nm/810nm Green/Red/Infrared - 532nm/670nm/810nm Yellow/Red/Infrared - 577nm/670nm/810nm

Four Wavelengths

Green/Yellow/Red/Infrared -532nm/577nm/670nm/810nm Green/Yellow/Red/Infrared -532nm/561nm/670nm/810nm

TECHNICAL SPECIFICATIONS

Max Power Output on Cornea 2W 1.5W 2W 0.7W 3W* Wavelength 532nm 561nm 577nm 670nm 810nm Mode of Operation Continuous Wave Laser Type Diode Pumped Solid-State Laser Optically Pumped Semiconductor Laser Diode Laser Exposure Duriation 10 to 3000ms, continuously variable (10 to 90s, continuously variable for 810nm only) Repeat Interval 10 to 3000ms, and single SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50% Duty Cycle Settings: 2.5%, 5%, 10%, 15%, 20%, 30%, 40%, 50% (25% and 31.3% for 810nm only) Pattern Scanning Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing) Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Safety Class Class 4 Aiming Laser Red laser dlode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Dlode and Crystal Dimensions LiGHTLas TruScan Pro Laser Console: 13 cm (H) × 40.5 cm (M) × 40 cm (D) Weight LiGHTLas TruScan Pro Laser Console: 88kg Complete System on Table: 130kg		GREEN 532nm	YELLOW 561nm	YELLOW 577nm	RED 670nm	INFRARED 810nm
Mode of Operation Continuous Wave Optically Pumped Semiconductor Laser Diode Laser Exposure Duration 10 to 3000ms, continuously variable (10 to 90s, continuously variable for 810nm only) Repeat Interval 10 to 3000ms, and single SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50% Duty Cycle Settings: 2.5%, 5%, 10%, 15%, 20%, 30%, 40%, 50% (25% and 31.3% for 810nm only) Pattern Scanning Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100, 200, 300, 400, 500 μm Safety Class Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) × 40.5 cm (W) × 40 cm (D) Complete System on Table: 140 cm (H) × 92 cm (W) × 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg		2W	1.5W	2W	0.7W	3W*
Laser Type Diode Pumped Solid-State Laser Optically Pumped Semiconductor Laser Diode Laser Exposure Duration 10 to 3000ms, continuously variable (10 to 90s, continuously variable for 810nm only) Repeat Interval 10 to 3000ms, and single SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50% Duty Cycle Settings: 2.5%, 5%, 10%, 15%, 20%, 30%, 40%, 50% (25% and 31.3% for 810nm only) Pattern Scanning Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Scanning Patterns: 100, 200, 300, 400, 500 μm Safety Class Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) x 40.5 cm (W) x 40 cm (D) Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Wavelength	532nm	561nm	577nm	670nm	810nm
Exposure Duration 10 to 3000ms, continuously variable (10 to 90s, continuously variable for 810nm only) Repeat Interval 10 to 3000ms, and single SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50%	Mode of Operation	Continuous Wave				
Repeat Interval 10 to 3000ms, and single SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50% Duty Cycle: 2.5%, 5%, 10%, 15%, 20%, 30%, 40%, 50% (25% and 31.3% for 810nm only) Pattern Scanning Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Scanning Patterns: 100, 200, 300, 400, 500 μm Safety Class Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) x 40.5 cm (W) x 40 cm (D) Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Laser Type	Diode Pumped Solid-State Laser		' '	Diode Laser	
SP-Mode® Setting SP-Mode® Duty Cycle: 2.5% - 50%	Exposure Duration	10 to 3000ms, continuously variable (10 to 90s, continuously variable for 810nm only)				
Duty Cycle Settings: 2.5%, 5%, 10%, 15%, 20%, 30%, 40%, 50% (25% and 31.3% for 810nm only) Pattern Scanning Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Scanning Patterns: 100, 200, 300, 400, 500 μm Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) × 40.5 cm (W) × 40 cm (D) Complete System on Table: 140 cm (H) × 92 cm (W) × 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Repeat Interval	10 to 3000ms, and single				
Spot Size Single Spot: 50-1000 μm Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Scanning Patterns: 100, 200, 300, 400, 500 μm Safety Class Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) × 40.5 cm (W) × 40 cm (D) Complete System on Table: 140 cm (H) × 92 cm (W) × 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	SP-Mode® Setting					
Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing) Scanning Patterns: 100, 200, 300, 400, 500 μm Safety Class Class 4 Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) × 40.5 cm (W) × 40 cm (D) Complete System on Table: 140 cm (H) × 92 cm (W) × 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Pattern Scanning	Single, Straight, Square, Triangle, Circle, Arc, Curve and Equal Square Spacing				
Aiming Laser Red laser diode (635nm), under 1.0mW continuously variable Cooling System Fan cooled and TEC's for Laser Diode and Crystal LIGHTLas TruScan Pro Laser Console: 13 cm (H) x 40.5 cm (W) x 40 cm (D) Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Spot Size	Limited Scanning Patterns: 100 μm (only Straight, Square, Triangle, Equal Square Spacing)				
Cooling System Fan cooled and TEC's for Laser Diode and Crystal LIGHTLas TruScan Pro Laser Console: 13 cm (H) x 40.5 cm (W) x 40 cm (D) Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Safety Class	Class 4				
Dimensions LIGHTLas TruScan Pro Laser Console: 13 cm (H) x 40.5 cm (W) x 40 cm (D) Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Aiming Laser	Red laser diode (635nm), under 1.0mW continuously variable				
Complete System on Table: 140 cm (H) x 92 cm (W) x 63 cm (D) Weight LIGHTLas TruScan Pro Laser Console: 88kg	Cooling System	Fan cooled and TEC's for Laser Diode and Crystal				
	Dimensions					
	Weight					

*2W with scan slit lamp delivery

LASER CONTACT LENSES: Ocular, Mainster 165 PRP (M = 1.96x) | Volk, Super Quad 160 (M = 2.00x) | Ocular, Latina SLT Gonio (M = 1.00x) | Ocular, Mainster Standard (M = 1.05x) | Ocular, Fundus Laser (M = 1.08x) | Ocular, Karichoff Laser (M = 1.08x) | Ocular, Three Mirror Universal (M = 1.08x) | Ocular, Mainster Wide Field (M = 1.50x) | Ocular, Mainster Ultra Field (M = 1.90x) | Rodenstock Schlegel Panfundoscope (M = 1.50x) | Volk, G-3 Goniofundus (M = 0.94x) | Volk, Area Centralis (M = 0.94x) | Volk, Trans Equator (M = 1.44x) | Volk, QuadrAspheric (M = 1.97x) | Volk, H-R Wide Field (M = 2.00x)

Optional Accessories

- TruLase integrated LIO
- Mobile cart
- Endoprobes & G-probe
- 3D mouse
- 7" remote control touchscreen





Specifications are subject to change without notice. LIGHTMED devices are made strictly in accordance with the international laser safety regulations and standards: EN60601-1, EN60601-1-1-2, EN60601-2-22, IEC 60852-1, IEC 60852-1



1130 Calle Cordillera | San Clemente, CA 92673 | USA T: 949-218-9555 | F: 949-218-9556 | sales@lightmed.com www.lightmed.com



LIGHTLas TruScan Pro

with SP-Mode®

PATTERN SCANNING PHOTOCOAGULATOR

TruScan Pro



SUPERIOR PERFORMANCE WITH FOUR WAVELENGTHS



CUSTOMIZABLE PATTERNS AND SPOT SIZES

levels with the fastest scanning system on the market.



SIMPLE INTEGRATION TO SUIT ALL PRACTICE REQUIREMENTS


Available with a choice of up to 4 customizable wavelengths, the LIGHTLas TruScan Pro is designed for traditional use or highly specialized needs in all types of clinical settings. Its modular, compact and portable console helps meet and exceed treatment goals.

Designed for complete versatility and ultimate performance, LIGHTLas TruScan Pro is the only laser in its class with a choice of four customizable wavelength options.

Customizable Wavelength Options Enhance Your Investment

LIGHTLas TruScan Pro is the only modular laser on the market that allows a physician to obtain the system in single-wavelength form and add up to 3 additional wavelengths in the future. Available wavelengths include green 532nm, yellow 561nm or 577nm, red 670nm and infrared 810nm.

For a complete list of custom wavelength configurations, refer to back of the brochure.

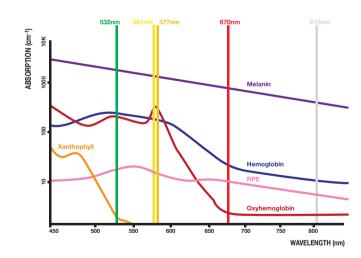

Green 532nm — Proven to Withstand High-Activity Usage with Superior Long-Term Performance

- Clinically Proven: Ideal for a large variety of retinal conditions with melanin as the target chromophore
- Immediate Visible Tissue Response: Allows precise administration of laser power

Yellow 561nm or 577nm — The Gold Standard in Laser Therapy

- Accurate Targeting: Selectively absorbed by melanin rich cells of RPE reducing retinal toxicity
- Closer Approach: Significantly increases the safety margins for macular treatment with immediate access to fovea when compared to other wavelengths
- Reduced Power: Typically requires 50% less power to achieve the same therapeutic effects as conventional green laser photocoagulation

Red 670nm — Optimal for Choroidal Photocoagulation


- Minimized Absorption: Nominal hemoglobin absorption for exceptional penetration of moderate vitreous hemorrhage
- Precise Application: Preferable for selective treatment of choroidal vessels without coagulation of retinal vessel

Infrared 810nm — Broad Range of Treatment Modalities


- Deep Penetration: Well absorbed by melanin and superior for excellent penetration into choroidal region
- Patient Comfort: The nearly invisible light in the infrared spectrum increases patient comfort
- Treatment of Choice: Used as laser of choice for Retinopathy of Prematurity (ROP), certain types of retinal lesions and in some cases of glaucoma

Consistent Power and Control

- Beneficial Features: Continuous parfocal variable, clinically guided spot size controls, excellent optical design, dual port system and laser cavity technology assure the highest standards of energy density, speed and convenience
- Range of Treatment Spot Sizes: Easily adjust treatment patterns for shape spacing, 360 degree rotation, and separation varying from 50 μm to 1000 μm (100 μm to 500 μm in pattern scanning mode) for easy application of confluent shots

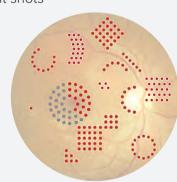

Position of scan patterns for simulation purposes only. Limited scanning patterns available at 100 μ m. See technical specifications for details.

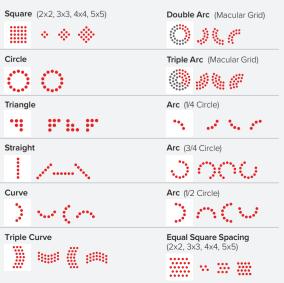

Advanced LCD Touch Screen Interface

LIGHTLas TruScan Pro increases treatment speed, safety, and convenience with a large selection

of scanning patterns. Enhance conventional treatment outcomes and your patients' comfort

- Built-In Patient Reporting: After treatment, parameters are automatically logged into patient reports with an integrated computer to export patient information files
- Fully Intuitive Platform: Adjustable treatment parameters and functions, storage capability for preferred treatment settings, automated lens calculation and more



Ergonomic wheelchair accessible table



Wireless Foot Pedal with Power Control

• Ergonomically Designed:
Foot pedal allows
hands-free operation for
increased visual focus.
A simple tap adjusts
treatment power settings
quickly and easily.



TruLase Laser Indirect Ophthalmoscopes (LIO*)

Precise Viewing:
 Integrated single or
 multi-wavelength LIOs
 provide unique control of
 aperture size and
 spot positioning.



Joystick for Optimal Precision

 Accurate Results: Integrated microjoystick provides excellent fingertip control and accuracy of treatment spots and patterns to improve outcome and speed.



for Optimal Portable Remote Control

• Easy Integration:
The console detaches
easily for portable use in
any room with LIGHTMED
delivery devices, and
includes a 7 inch remote
control touchscreen for
ultimate versatility.

Integrated Heads Up Display (HUD)

• Safer Treatment: HUD is built into the pattern scanning slit lamp to ensure focus remains on the patient.

Wide Range of Probes*

- Endoprobes available in boxes of 5 as Straight or Angled for sizes 20G, 23G and 25G.
- G-Probe avaliable in boxes of 5 as an additional glaucoma management device.

Compact Mobile Cart*

• Uniquely Portable: The laser console easily detaches for moving between rooms.



3D Mouse for Comfortable Control*

- Easy to Adjust Patterns: Allows users to easily control and select desired patterns with functions including rotation, pattern group selection, spacing and radius.
- Simple Touch to Adjust Parameters: Allows users to adjust power, duration and interval settings

* Optional accessories

